



WARNINGS

- Do not attempt to sterilize the equipment as this may cause damage - clean by wiping with a damp cloth
- Do not submerge in water
- For external use only - keep the camera a safe distance away from the sterilized zone
- Ensure all parts are secure before use
- Disable any screen savers or computer power saving options before use
- Computer sign in must have administrator rights for the SurgiCam HD to work as designed

ADDITIONAL HELP & RESOURCES

- Refer to the full manual on the enclosed USB Drive
- Check the support section on our website at www.surgitel.com/help
- Download the newest SurgiCam Software at www.surgitel.com/surgicam
- Contact SurgiTel Customer Service at 1-800-959-0153 or support@surgitel.com

— **SurgiTel®** —



SurgiCam® HD QUICK-START GUIDE

SurgiTel®
A Division of **General Scientific Corporation**
77 Enterprise Drive • Ann Arbor, MI 48103-9503
E-mail support@surgitel.com • www.surgitel.com

Doc No. 26427A

Full Manual Available on Enclosed USB Drive

SETUP

SOFTWARE

1. Install the SurgiCam HD software from the enclosed USB drive to your computer
2. Plug the SurgiCam into an open USB port before starting the software

HARDWARE

1. Attach the SurgiCam HD to your loupes, headband or stationary arm
2. Adjust the iris of the SurgiCam HD by turning the outer lens (**Figure 1**)
 - A more closed iris increases depth-of-focus while a more open iris maximizes the amount of light received
 - Apply the best setting for your current lighting conditions before recording



Figure 1
Iris Adjustment

RECORDING

1. Open the SurgiCam HD Software after the camera is plugged in - you will see a live feed from the camera in the Display Panel (see **Figure 2**)
2. Enter information for the recording in the Information Panel ("File Name" is required)
3. Click the record button in the Video Control Panel to begin recording video - or the snapshot button to take a snapshot
4. Click the record button again to stop recording

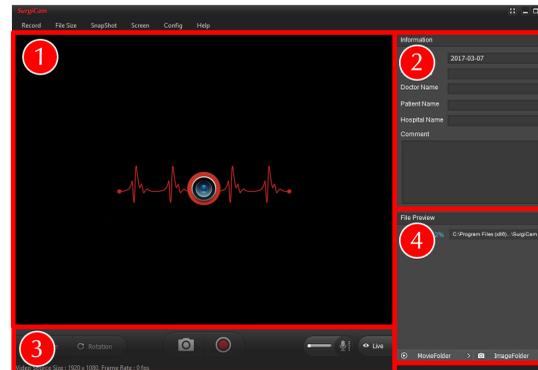


Figure 2 - SurgiCam HD Software

1. Display Panel
2. Information Panel
3. Video Control Panel
4. File Preview Panel

VIEWING

1. Click any of the finished recordings or snapshots which are displayed in the File Preview Panel (see **Figure 2**)
2. Your media will appear in the Display Panel - play videos by clicking the play button in the Display Panel
 - You may view the recorded media using the computer operating system by clicking the MovieFolder or ImageFolder buttons in the File Preview Panel